Mã: H42A; H42B; H42C

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Code: H42A; H42B; H42C

- Product name: Diagnostic Kit for differential detection of THC/MET/MOR/MDMA/KET
- Model name: dfu EASY TEST DOA 5-1
- Packaging: 15 tests / box; 20 tests / box; 25 tests / box
- **Granted circulation number: 250000569/PCBB-HCM**

COMPOSITION OF KIT

Composition of 1 kit	Product code / Packing size		
	H42A 15 tests / kit	H42B 20 tests / kit	H42C 25 tests / kit
Dip-card in aluminum pouch with desiccant (ea)	15	20	25
Plastic cup (ea)	15	20	25
Instruction for use (ea)	1	1	1

DETECTION THRESHOLD

Substance	Concentration (ng/mL)
11-nor- Δ^9 -THC-9 COOH (THC)	50
Methamphetamine (MET)	500
Morphine (MOR)	300
Methylenedioxymethamphetamine (MDMA)	500
Ketamine (KET)	1000

5-indicator drug test – A quick and effective solution to determine drug use status in the body.

DIAGNOSIS F BIOTECHNOLOGY PRODUCTION COMPANY LIMITED

5th Floor, Factory no.2, Lot I-3b-1, Street N6, Saigon Hi-tech Park, Tan Phu Ward, Thu Duc City, Ho Chi Minh City, Vietnam.

+123-456-7890

www.dfu.com.vn

dfu

Diagnosis F co.,Ltd.

THC/MET/MOR/MDMA/KET

INTRODUCTION:

Drug addiction is becoming more and more common and alarming. Normally, it is difficult to identify a drug addict unless they are in an addiction. However, the THC/MET/MOR/MDMA/KET urine test is the fastest, safest and most effective form of testing to identify and screen drug users or addictive substances.

dfu EASY TEST DOA 5-1

Check fast – Accurate results

HOT ISSUE?

EFFECTS OF DRUGS:

Alarming 67% of traffic accidents are related to drugs.

45% of businesses have labor accidents due to employees using stimulants.

Individuals: Memory loss, organ damage, death due to drug shock.

Businesses: Loss of reputation, reduced productivity, legal risks.

Society: Economic burden, loss of security and order.

" dfu EASY TEST DOA 5-1 " CONVENIENT - FAST SOLUTION

- ✓ Early detection of 5 dangerous addictive substances in one test.
- ✓ Save 80% of time compared to lab testing.
- ✓ Prevent risks before it's too late.

EASY TO USE

Step 1

Allow the test tray, urine specimen and/or control specimen to reach room temperature (15 - 30°C) prior to testing.

Step 2

Remove the test tray from the aluminum pouch and use immediately.

Step 3

Following the arrow pointing towards the urine sample, immerse the test tray into the urine sample in the direction of the arrow, holding for at least 10 - 15 seconds. Do not let the urine touch the test tray.

Step 4

Place the test strip on a flat, non-absorbent surface, start the timer and wait for the red lines to appear.

Step 5

Read results after 5 minutes. Do not read results after 10 minutes.

dfu EASY TEST DOA 5-1

Tetrahydrocannabinol (THC)

is the main active ingredient in the Marijuana plant. Using THC will cause serious consequences such as memory loss, reduced learning ability, loss of body control, and behavioral disorders.

Methamphetamine (MET)

Methamphetamine - a powerful stimulant, highly addictive and affects the central nervous system. MET stimulates the brain to secrete large amounts of dopamine and dopamine makes the user feel excited but not according to the body's natural mechanisms. This makes the person crave the experience again, leading to addiction.

Morphine (MOR)

Opiates are a large group of substances that control pain by inhibiting the central nervous system. In addition to their pain-relieving effects. This group also causes very serious consequences such as drug dependence. If you want to relieve pain, you have to increase the dose of the drug, which is addictive and dangerous.

Methylenedioxymethamphetamine (MDMA)

is ecstasy, an amphetamine-like stimulant and hallucinogen. MDMA causes euphoria and disinhibition and enhances physiological sensations, empathy, and feelings of closeness between individuals. The effects begin within 30 to 60 minutes and typically last for 4 to 6 hours. Repeated, long-term use can cause problems similar to amphetamines, including dependence, schizophrenia, and cognitive impairment in some people.

Ketamine (KET)

is a medically used anesthetic, but is abused as a narcotic with hallucinogenic and temporary paralytic effects. The side effects of KET include memory loss, nervous system damage, and the risk of addiction. Long-term use can lead to kidney failure and mental disorders.