dfu EASY TEST CDV/CAV2

Canine Distemper Virus and Adenovirus type 2 Antigen Test test
in eye mucus and nasal discharge of dog

dfu

For Veterinary Use Only

INTENDED USE

The dfu EASY TEST CDV/CAV2 is a rapid chromatographic immunoassay for the qualitative detection of Canine distemper virus (CDV) and Adenovirus type 2 in eye mucus and nasal discharge of dog.

PRINCIPLE

The dfu EASY TEST CDV/CAV2 is a qualitative, membrane-based immunoassay for the detection of CDV and Adenovirus type 2 in eye mucus and nasal discharge of dog. The membrane is pre-coated with anti-CAV2 monoclonal antibody on the T1 test line and anti-CDV monoclonal antibody on the T2 test line region of the cassette. During testing, the mixture of sample with assay buffer migrates upward from the sample well reacts with another CAV2 specific antibody-gold particle conjugates and CDV specific antibody-gold particle conjugates on conjugate pad and continue to move on the membrane by capillary forces to react with specific antibody on the membrane and generate red lines on the test lines. Presence of these red lines indicate a positive result, while their absence indicates a negative result. A red line will always appear at the control line region to serve as a proper performance control. With this dfu EASY TEST kit, the diagnostic reagent can detect CDV and CAV2 antigens quickly and simply within 10 minutes after adding the extracted sample.

KIT COMPONENTS

Reagent	10 tests/box
CDV/CAV2 device	10
Extraction solution (tube 0.5 ml)	10
Dropper	10
Swab	10
Instruction for use	1

PRECAUTIONS

- For veterinary use only
- The test kit should remain in the sealed pouch until ready for use.
- The test kit is sensitive to humidity and to heat.
- Do not smoke, eat or drink in areas where specimens or kit component are handled.
- All specimens and reagents should be considered potentially hazardous and handled in the same manner as an infection agent.
- Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are assayed.
- Never use reagents from another kit
- Discard the extraction solution if it contaminated with bacteria or mold.


SPECIMEN COLLECTION AND STORAGE

- Use eye mucus or nasal discharge of dog as sample
- Sample to be tested should be obtained and handle by standard methods for the collection of body secretions.
- After collecting the sample using swab, all the samples should be immediately extracted and tested. If necessary, they may be stored at 2 to 8°C for up to 2 days or at -20°C for longer periods.

TEST PROCEDURE

- Take a portion of secretions of eye mucus and nasal discharge by using a sterilized swab.
- Insert and mix the swab with assay solution in a tube until the samples have been dissolved. Then discard the swab.
- Place the dfu EASY TEST CDV/CAV2 device on a clean and flat surface.

- Add 3-4 drops of the mixture solution taken by the disposable dropper into the sample well of the device.
- Verify the test results within 10 minutes. Do not interpret after 10 minutes.

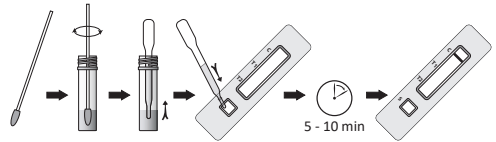INTERPRETATION OF RESULTS

- Positive:
 - CAV2 positive result: Two red lines appear in the T1 test line and control line (C)

- CDV positive result: Two red lines appear in the T2 test line and control line (C)

- CAV2 and CDV positive result: Three red lines appear in the test line (T1 and T2) and control line (C).

- Negative: When there is a red control line but no test line.

- Invalid: When there are red test lines but no control line, or there are no control line and test line.

- Regardless of CDV and CVA type 2 antigen presence, a control line should always appear. The control line is needed to check whether abnormal reaction occurs or not, so if there is no control line, re-test should be performed.

STORAGE AND EXPIRATION

- Store devices and test reagents at 2-30°C. Do not freeze. All components must be at room temperature (18-25°C) before running the test. Do not heat.
- Expiration date of this kit is 24 months after its manufacture date.

LIMITATION OF THE TEST

dfu EASY TEST CDV/CAV2 is designed for primary screening test of Canine Distemper virus and Adenovirus 2 antigen. This kit can provide fast and easy way to get a result, but do not completely exclude the possibility of false positive or false negative result caused by various factors. So, refer to the result of this kit, please make a final decision with clinical manifestation, other test results, and veterinarian's view, collectively.

dfu

Manufactured by
DIAGNOSIS F CO., LTD

Address: 5th floor, Factory no.02, Lot I-3b-1, Street N6, Saigon Hi-Tech Park, Tan Phu Ward, Thu Duc City, Ho Chi Minh City

Tel: (+84) 2871098734
Website: www.dfu.com.vn
Product Code: A15
MB049-Ver.02 (Rev.30/06/2025)

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