INTENDED USE

The dfu EASY TEST PEDV is a rapid chromatographic immunoassay for the qualitative detection of Porcine Epidemic Diarrhea Virus (PEDV) in porcine feces.

PRINCIPLE

The dfu EASY TEST PEDV is a qualitative, membrane-based immunoassay for the detection of PEDV in porcine feces. The membrane is pre-coated with PEDV-specific antibody on the test line region of the cassette. During testing, the mixture of feces sample with dilution buffer migrates upward from the sample well reacts with another PEDV-specific antibody-gold particle conjugates on conjugate pad and continue to move on the membrane by capillary forces to react with specific antibody on the membrane and generate a red line on the test line. Presence of this red line indicates a positive result, while its absence indicates a negative result. A red line will always appear at the control line region to serve as a proper performance control. With this dfu EASY TEST kit, the diagnostic reagent can detect PEDV antigens quickly and simply within 10 minutes after adding the extracted sample.

KIT COMPONENTS

Reagent	10 tests/box
PEDV device	10
Extraction solution (vial 1 ml)	10
Swab	10
Dropper	10
Instruction for use	1

PRECAUTIONS


- For veterinary use only
- The test kit should remain in the sealed pouch until ready for use.
- The test kit is sensitive to humidity and to heat.
- Do not smoke, eat or drink in areas where specimens or kit component are handled.
- All specimens and reagents should be considered potentially hazardous and handled in the same manner as an infection agent.
- Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are assayed.
- This dfu EASY TEST kit is made for primary screening. The samples which have been determined as positive should be re-tested by more precise, sophisticated methods to get the comprehensive final diagnosis from professional veterinarian.

SPECIMEN COLLECTION AND STORAGE

- Use porcine feces as samples
- Take the sample by pricking inside of feces deeply or pulling out directly through pig's anus using the sample collection swab.


TEST PROCEDURE

- Collect a diarrhea specimen from diarrhea feces or those in the anus using swabs for fecal specimen collection.
- Put the sample into the container that contains extraction solution and stir well the solution with a swab in order to extract the virus from the fecal sample thoroughly.
- Place the tube upright until the large particles go down. (20 sec.)
- Remove the dfu EASY TEST PEDV device from the pouch and place it on a clean and flat surface.
- Take the supernatant of sample solution using dropper, and then transfer 4 drops into the sample well of device.
- Verify the result within 5 minutes to 10 minutes.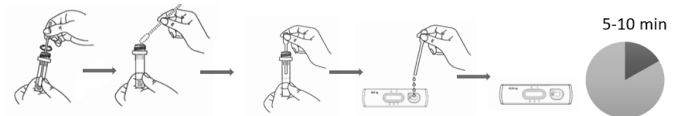

INTERPRETATION OF RESULTS

- Positive:** When there are a red control line and a red test line.

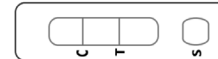- Negative:** When there is a red control line but no test line.

- Invalid:** When there is a red test line but no control line, or there are no control line and test line.

- * Regardless of PEDV presence, a control line should always appear. The control line is needed to check whether abnormal reaction occurs or not, so if there is no control line, re-test should be performed.

STORAGE AND EXPIRATION

- Store devices and test reagents at 2-30°C. Do not freeze. All components must be at room temperature (18-25°C) before running the test. Do not heat.
- Expiration date of this kit is 24 months after its manufacture date.

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