dfu EASY TEST FCoV Ab

Feline Coronavirus Antibody test in feline serum, plasma or whole blood

dfu

For Veterinary Use Only

INTENDED USE

Feline Coronavirus (FCoV) is a highly contagious intestinal infection that causes few problems in most infected cats. In some cats, however, the intestinal form of FCoV mutates and gives rise to feline infectious peritonitis (FIP), a disease that is generally fatal. It is estimated that up to 70% of cats, worldwide, are exposed to Feline Coronavirus (FCoV).

dfu EASY TEST FCoV Ab is a chromatographic immunoassay kit for rapid and qualitative detection of feline coronavirus antibody in feline serum, plasma or whole blood.

KIT COMPONENTS

Reagent	10 tests/box
FCoV Ab device	10
Assay solution (vial 1 ml)	10
Disposable 10µl capillary tube	10
Dropper	10
EDTA tube	10
Instruction Manual	1

PRECAUTIONS


- For veterinary use only
- The test kit should remain in the sealed pouch until ready for use.
- The test kit is sensitive to humidity and to heat.
- Do not smoke, eat or drink in areas where specimens or kit component are handled.
- All specimens and reagents should be considered potentially hazardous and handled in the same manner as an infection agent.
- Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are assayed.
- Never use reagents from another kit
- Discard the assay solution if it contaminated with bacteria or mold.

SPECIMEN COLLECTION AND STORAGE

- Use dog serum, plasma or whole blood as sample
 - Whole blood: Collect the blood into the tube contained anticoagulants such as heparin, citrate, or EDTA using standard clinical laboratory procedures. Anticoagulated whole blood samples should be tested within 24 hours of drawing. If delays are expected between sampling and testing, the sample should be stored either on ice or refrigerated (2~8°C), but should not be frozen
 - Serum: Collect the blood, allow the blood to clot, then centrifuge to separate the serum.
 - Plasma: Collect the blood into the tube contained anticoagulants such as heparin, citrate, or EDTA. Centrifuge the blood and separate the plasma.
2. All specimens should be tested as soon as they are prepared. If necessary, they may be stored at 2~8°C for up to 24 hours or at -20°C for longer periods.

TEST PROCEDURE

- Place the dfu EASY TEST FCoV Ab device on a clean and flat surface.
- Add 10µl of the specimen, using a disposable capillary tube provided to the assay solution tube. Gently stir the mixture solution with the capillary tube to make sure the suitable mixing of specimen with assay solution then remove the capillary tube.
- Take the mixture solution using a dropper, and then transfer 3 drops into the sample well of the device.
- Verify the result at 10 minutes. Do not read the result after 15 minutes.

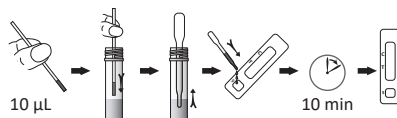
INTERPRETATION OF RESULTS


- Negative result: ONLY one band in the control line (C).

- Positive result: Two bands are appeared in the test line (T) and control line(C)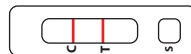

- Invalid result: If at 10 minutes, the red color band does not appear in the control line (C), even if any shade of a pink-to-red test line (T) appears, the result is considered invalid. If the test is invalid, a new test should be performed with a new sample and a new test device.

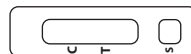STORAGE AND EXPIRATION

- Store devices and test reagents at 2-30°C. Do not freeze. All components must be at room temperature (18-25°C) before running the test. Do not heat.
- Expiration date of this kit is 24 months after its manufacture date.

LIMITATIONS OF THE TEST

dfu EASY TEST FCoV Ab is designed for primary screening test antibodies against feline coronavirus in the whole blood, serum or plasma of cat. This kit can provide fast and easy way to get a result, but do not completely exclude the possibility of false positive or false negative result caused by various factors. So, refer to the result of this kit, please make a final decision with clinical manifestation, other test results, and veterinarian's view, collectively.

dfu

Manufactured by
DIAGNOSIS F CO., LTD
Address: 5th Floor, Factory no.02, Lot I-3b-1, Street N6, Saigon Hi-Tech Park,
Tang Nhon Phu Ward, Ho Chi Minh City, Vietnam

Tel: (+84) 2871098734
Website: www.dfu.com.vn
Product Code: A07
MB041-Ver.02 (Rev. 08/04/2026)

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