dfu EASY TEST FPV

Feline Parvovirus (FPV) antigen rapid test in cat feces.

dfu

For Veterinary Use Only

INTENDED USE

The dfu EASY TEST FPV is an immunoassay kit for rapid diagnosis, qualitative detection of Feline Parvovirus (FPV) antigen in feces of cat.

PRINCIPLE

dfu EASY TEST FPV is a chromatographic immunoassay kit for the qualitative detection of Feline Parvovirus (FPV) antigen in feces of cat. The cat feces sample will be extracted with extraction solution and this sample solution was added to the sample well. During test, if FPV antigens are present in the sample solution, these antigens will bind to FPV specific antibody-gold particle conjugates and move on the membrane by capillary forces, and then shows a red line on the test line (T) due to the binding with FPV specific antibodies which are already coated on the membrane. Appearance of this red line indicates a positive result, while its absence indicates a negative result. A red line will always appear at the control line (C) to serve as a suitable performance control. The test device can quickly and simply detect the FPV antigen within 10 minutes after dropping the sample solution.

KIT COMPONENTS

Components	10 tests/kit
FPV device	10 devices
Extraction solution (1.0 mL/vial)	10 vials
Dropper	10 ea
Sample collection swab	10 ea
Instruction for use	1 sheet

PRECAUTIONS


- For *in-vitro* veterinary diagnostic use only.
- Read this instruction for use thoroughly and follow all steps strictly for successful use of the product.
- Make sure to read the results on time.
- Do not use test device and reagents after expiration date, discard the extraction solution if it is contaminated with bacteria or mold.
- The test device is sensitive to humidity and heat so it should be stored in a sealed pouch until ready for use. Use the test device immediately after opening pouch.
- Wear personal protective equipment such as lab coat, goggle, and disposable gloves while performing the test. Wash hands thoroughly afterwards.
- Handle all specimens, test device safely and in accordance with relevant regulations because of the risk of infection.
- After using, all wastes should be sterilized with high-pressure steam at 121°C for ≥ 15 minutes or comparable methods.
- Do not smoke, eat or drink in areas where specimens or kit components are handled.
- Never use reagents from another kit.

SPECIMEN COLLECTION AND STORAGE

1. Use feces of cat as sample.
2. Collect a feces sample by spreading the swab tip to the fecal points (or pricking inside of feces deeply), or putting the swab tip directly into the rectum of cat.
3. After the feces sample collection, the sample should be extracted with extraction solution and tested immediately. If the sample is not tested immediately, it should be stored at 2 - 8°C within 48 hours or frozen (below -20°C) for a longer time.

TEST PROCEDURE

1. Place all components and samples at room temperature within 15 - 30 minutes before test.
2. Use a collection swab to collect a feces sample.
3. Put the sample swab into extraction solution tube and stir well to extract the virus from the fecal sample thoroughly, then discard the swab (Note: Do not over-extract the fecal sample to prevent blocked flow).
4. Place the sample solution tube upright to deposit sediment for 30 - 60 seconds.
5. Release the test device from the aluminum pouch and place it on a clean, dry, flat surface.
6. Take the supernatant of the sample solution using dropper and add 3 - 4 drops (approx. 100 µL) into the sample well.
7. Read the result at 10 minutes. Do not read the result after 10 minutes.

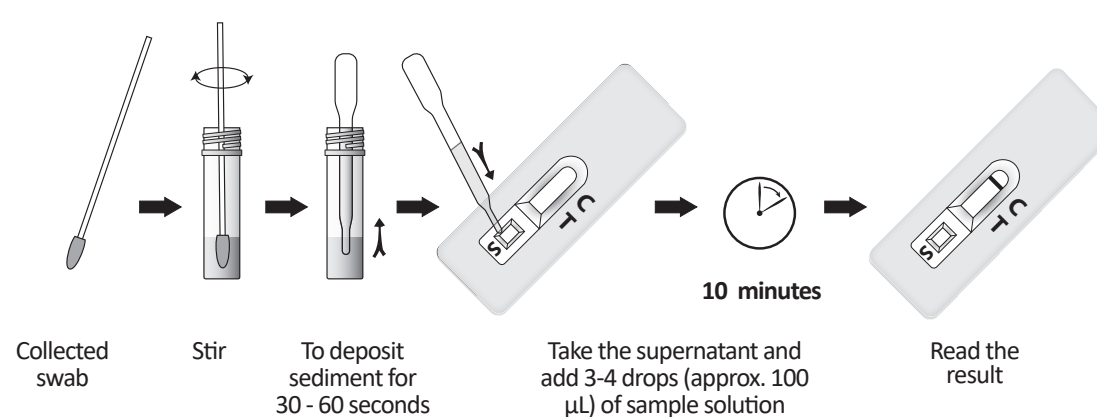INTERPRETATION OF RESULTS

1. **Positive:** Two red lines are appeared at the test line (T) and control line (C) in the result window.

2. **Negative:** One red line is appeared at the control line (C) in the result window.

3. **Invalid:** The red line does not appear at the control line (C) in the result window.

- The control line (C) is necessary in checking whether the reaction occurs abnormally or not. Therefore, if the control line does not appear, repeat the test with another test device.

STORAGE AND EXPIRATION

- Store the test kit at 2 - 30°C, do not freeze or heat.
- Expiration date: 24 months from manufacturing date.

LIMITATION OF THE TEST

The dfu EASY TEST FPV is manufactured for diagnostic and screening detection of FPV antigen in feces of cat, so do not completely exclude the possibility of false positive or false negative result. The final result needs to be based on clinical symptoms or higher assay techniques and the veterinarian's experience.

dfu

Manufactured by

DIAGNOSIS F CO., LTD

Address: 5th Floor, Factory No.02, Lot I-3b-1, Street N6, Saigon Hi-Tech Park, Tang Nhon Phu Ward, Ho Chi Minh City, Vietnam.

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