INTENDED USE

The dfu EASY TEST FIV Ab/FeLV is a rapid chromatographic immunoassay for the qualitative detection of the antibodies against Feline immunodeficiency virus (FIV) and leukemia virus antigen using feline serum, plasma or whole blood.

PRINCIPLE

The dfu EASY TEST FIV Ab/FeLV is a qualitative, membrane-based immunoassay for the detection of FIV Antibody/FeLV antigen in feline serum, plasma or whole blood. The membrane is pre-coated with FIV antigen and FeLV antibody in the test line and with anti-mouse antibody in the control line. And also, FIV antigen and FeLV-specific antibody is conjugate to the colloidal gold particles. This conjugate is placed on polyester or glass fiber as conjugate pad. When the sample is dropped in to the sample well, the solubilized conjugate migrates with the sample by passive diffusion and both the conjugate and sample come into contact with the antibody that immobilized onto the nitrocellulose. If the sample contains FIV antibody and/ or FeLV antigen, the result are visible as red line with 10 minutes in the test lines on the membrane.

KIT COMPONENTS

Reagent	10 tests/kit
FIV Ab/FeLV device	10 devices
Assay solution (4mL/bottle)	1 bottle
Disposable capillary tube	10 ea
EDTA tube	10 tubes
Instruction for use	1 sheet

PRECAUTIONS

- For veterinary use only
- The test kit should remain in the sealed pouch until ready for use.
- The test kit is sensitive to humidity and to heat.
- Do not smoke, eat or drink in areas where specimens or kit component are handled.
- All specimens and reagents should be considered potentially hazardous and handled in the same manner as an infection agent.
- Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are assayed.
- Never use reagents from another kit
- Discard the assay solution if it contaminated with bacteria or mold.

SPECIMENCOLLECTION AND STORAGE

- Use Feline serum, plasma or whole blood as sample
 - Whole blood: Collect the blood into the tube contained anticoagulants such as heparin, citrate, or EDTA using standard clinical laboratory procedures. Anticoagulated whole blood samples should be tested within 24 hours of drawing. If delays are expected between sampling and testing, the sample should be stored either on ice or refrigerated (2-8°C), but should not be frozen
 - Serum: Collect the blood, allow the blood to clot, then centrifuge to separate the serum.
 - Plasma: Collect the blood into the tube contained anticoagulants such as heparin, citrate, or EDTA. Centrifuge the blood and separate the plasma.
2. All specimens should be tested as soon as they are prepared. If necessary, they may be stored at 2-8°C for up to 24 hours or at -20°C for longer periods.


TESTPROCEDURE

- Place the dfu EASY TEST FIV Ab/FeLV device on a clean and flat surface.
- Load 1 drop (10µl) of specimen into the sample well (S).
- Add 3 drops of assay solution into the sample well (S)
- Verify the test results within 10 minutes. Do not interpret the result after 10 minutes.


INTERPRETATION OF RESULTS

- Negative Results:** The presence of only one red band (control band) within the result window on both of the FIV and FeLV test regions indicates a negative result.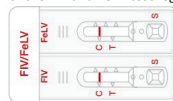

- Positive Result:**

FIV positive result: The presence of two red bands ("T" and "C") within the result window on the FIV test region, and the presence of only one red band ("C") within the result window on the FeLV test region, regardless of which band appears first, indicates a positive result of FIV Ab

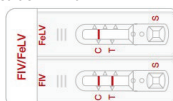
FeLV positive result: The presence of two red bands ("T" and "C") within the result window on the FeLV test region, and the presence of only one red band ("C") within the result window on the FIV test region, regardless of which band appears first, indicates a positive result of FeLV

FIV and FeLV positive result: The presence of two red bands ("T" and "C") within the result window on the both of FIV and FeLV test regions respectively, regardless of which band appears first, indicates a positive result of FIV Ab and FeLV

- Invalid result:** If at 10 minutes, the red color band does not appear in the control line (C), even if any shade of a pink-to-red test lines (T) appears, the result is considered invalid. If the test is invalid, a new test should be performed with a new sample and a new test device.

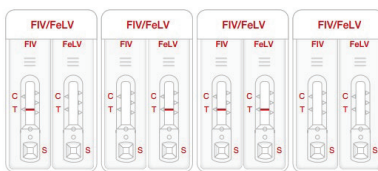STORAGE AND EXPIRATION

- Store devices and test reagents at 2-30°C. Do not freeze. All components must be at room temperature (18-25°C) before running the test. Do not heat.
- Expiration date of this kit is 24 months after its manufacture date.

LIMITATION OF THE TEST

dfu EASY TEST FIV Ab/FeLV is designed for primary screening test of antibodies of feline immunodeficiency and/ or antigen of feline leukemia virus infection. This kit can provide fast and easy way to get a result, but do not completely exclude the possibility of false positive or false negative result caused by various factors. So, refer to the result of this kit, please make a final decision with clinical manifestation, other test results, and veterinarian's view, collectively.

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